

General information:

Before use, the sample should be stored at 2-8°C. The first 2 drops should be dispensed and discarded.

- The dropper bottle is now ready to use to apply a sample drop for testing.
- After each use, be sure to wipe the dropper tip and inside of cap to keep them clean.

General information:

Before use, the sample should be stored at 2-8°C. The first 2 drops should be dispensed and discarded.

The dropper bottle is now ready to use to apply a sample drop for testing.

5. Dates and Submission of Test Results

Testing Periods: Please refer to the sample labels. Results can be submitted anytime within the testing period indicated on the sample labels.

Please submit your results electronically to ESfEQA on <https://teqa.esfeqa.eu>.

Contact your local distributor of ESfEQA programs or ESfEQA directly if you need assistance for registration in TEQA. Alternatively, though not preferred, use the result form provided on the ESfEQA homepage. In both cases, please indicate the instrument and method used for the analysis of the samples.

6. Deadline for Data Submission

The deadlines for submitting results are indicated on the sample labels (time zone GMT +1).

7. Reports and Certificates

The data will be evaluated by ESfEQA. The individual laboratory reports and certificates can be retrieved online at <https://teqa.esfeqa.eu>.